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Cera-Etch

Hydrofluoric Acid 9%

Cera-etch porcelain is a hydrofluoric acid that has gel properties and has been prepared with a concentration of %9 for oral intestinal repair of porcelain.

Ingredient:

Hydrofluoric acid, authorized dye material, gel base

Application:

porcelain Etch is to be used for etching fractured dental porcelain and for etching porcelain veneers, crowns or inlays in the office. Also use it for etching porcelain crowns before bonding orthodontic brackets.

Advantages:

Hydrofluoric acid gives off a vapor that irritates respiratory passageways and on a chronic basis can damage them. Morvabon Porcelain Etch is buffered, gelled and dispensed in a way that minimizes these problems.

Precautions:

- ▶ Carefully read and understand instructions before using. It's necessary to use rubber-dam while working with **Cera-Etch**.
- ▶ Use eye protection, mask and gloves when using **Cera-Etch**
- ▶ Avoid contact with oral soft tissue, eyes or skin
- ▶ If accidental contact with soft tissues, eyes or skin occurs, immediately flush with plenty of water
- ▶ In case of consumption drink copious amount of water.
- ▶ Do not induce vomiting. Seek medical advice
- ▶ Do not inject this product
- ▶ For dental use only
- ▶ Operator, assistant and patient must wear protective eyewear
- ▶ Replace luer lock caps on syringes when not in use.
- ▶ For professional dental use only.
- ▶ Be careful not to corrode steel.
- ▶ Use rubber-dam while working with the product.
- ▶ If swallowed: Wash mouth with water. Do not induce vomiting. Take water and then go to medical centers.
- ▶ Do not use in similar cases.
- ▶ At the end of the work, the remainder of the material must be completely removed from the patient's mouth.

Warning:

- ▶ Do not use after expiration date
- ▶ Keep out of reach of children
- ▶ Protect from direct sunlight.
- ▶ Store in the original container at room temperature.

Direction of Use:

- ▶ Freshen the porcelain surface to be repaired with a high speed diamond bur. If a previous repair is present,
- ▶ remove all old composite to a fresh porcelain surface.
- ▶ Remove luer lock caps from syringes.
- ▶ For optimum control, hold syringes in the palm of the hand.
- ▶ Isolate/protect adjacent tissues and restorations.
- ▶ Before applying, dispense a small sample of etchant From tip onto a mixing pad to ensure evenness and to become familiarized with flow rate
- ▶ Express an even coating of Porcelain Etch onto prepared porcelain Etch porcelain one minute.
- ▶ Assistant should hold vacuum tip adjacent to the surface while the dentist dispenses the Porcelain Etch.
- ▶ This ensures removal of any irritating vapors while acid reacts on porcelain (one minute).
- ▶ Remove Morvabon porcelain etch by placing the vacuum tip next to the surface and then rinsing with water spray.
- ▶ Rinse thoroughly and dry. Porcelain should have a frosted appearance. Repeat etching if not.

Contraindication:

- ▶ This product is not contraindicated for children, pregnant women, the elderly and other specific diseases and can be used for all people who are not allergic to the active substance

 Consult instructions for use	 Indicates the batch code	 Catalogue number	 Indicates the date after which the medical device not to be used.	 Indicates a medical device that needs protection from light sources.	 Indicates a medical device that needs to be protected from moisture.	 Indicates the Temperature Limit
 Do not use if Package is Damaged	 Indicates the medical device manufacturer (Name & Address)	 Indicates a medical device that has not been subjected to a sterilization process.	 Indicates the device intended for use by professional person only.	 Indicates that the medical device can cause discomfort and reddening, irritation, or blistering of the skin	 Indicates that the product has a chemical compound capable of metal corrosion or skin corrosion	

 Produced by Morvabon Co.
www.morvabon.com
info@morvabon.com
Made in IRAN