MORVABON

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MEDICAL
EQUIPMENT
DEPARTMENT

Reg. No. at IRAN Medical
equipment department: 1855

manufacturer

Pardis Morvabon Pharmed Co.:

Head Office: Unit 154, 7nd floor, C block,
Borjsaz Bd, Azadi Street, Tehran, Province, Iran

Tel: 0098 21 6658 1438

Fax: 0098 21 6690 8819

citric acid solution %40

citric acid solution 40%

To wash and prepare the root canal

Composition:

citric acid, water, stabilizer

Intended use:

- ▶ Root canal preparation
- ▶ Removal of debris and smear layer
- ▶ Root canal cleaning and disinfection
- ▶ It separates the smear layer from the root canal walls and exposes the dentine tubules.
- ▶ Help with flooding in the root canal

Advantages:

- ▶ Due to its more acidic pH, it is more effective than EDTA in removing the smear layer
- ▶ Citric acid solution is able to remove the smear layer after 60 seconds.
- ▶ Improves mechanical cleaning of the root canal.
- ▶ This product is a non-toxic substance for the human body.

Precautions:

- ▶ carefully read and understand instructions before using.
- ▶ It is necessary to use a rubber-dam while working with this product.
- ▶ use eye protection, mask and gloves when using citric acid solution.
- ▶ Do not use for patients who are highly allergic to the components of this product.
- ▶ Avoid contact with oral soft tissue, eyes or skin.
- ▶ In case of contact with soft tissues, eyes and skin, immediately wash the area with sufficient amount of water.
- ▶ In case of swallowing, drink plenty of water. Do not induce vomiting and consult a doctor.
- ▶ Do not inject this product.
- ▶ To be used only for dental purposes.
- ▶ Remove excess amounts from the patient's mouth.

- ▶ Be careful when using this product on children due to the risk of it becoming too soft.
- ▶ Do not use sodium hypochlorite immediately after washing with citric acid or vice versa
- ▶ When cleaning the root canal, do not use a strong flow of liquid, because it may cause the product to reach the periapical tissues and cause their irritation.

warning:

- ▶ Do not use after expiration date.
- ▶ Keep out of reach of children.
- ▶ Keep the product away from light.
- ▶ Store in the original packaging at a temperature of 25 °C.

Direction for use:

- ▶ Shake the product before use.
- ▶ Using a syringe with a needle, introduce 40% citric acid into 2/3 of the length of the root canal. Introduce the preparation using a syringe (eg Lore-Lac type) with a needle attached to the canal to ensure isolation.
- ▶ Flush 35 ml of the product after each mechanical canal dilation, preferably with other products intended for canal flushing. After washing the canal, aspirate the liquid and its contents. After rinsing the root canal, the excess amount in the syringe should be thrown away, do not pour back into the container.

Contraindication:

- ▶ This product is not contraindicated for children, pregnant women, the elderly and other specific diseases and can be used for all people who are not allergic to the active substance

					
Consult instructions for use	Indicates the batch code	Catalogue number	Indicates the date after which the medical device not to be used.	Indicates a medical device that needs protection from light sources.	Indicates a medical device that needs to be protected from moisture.
					
Do not use if Package is Damaged	Indicates the medical device manufacturer (Name & Address)	Indicates a medical device that has not been subjected to a sterilization process.	Indicates the device intended for use by professional person only.	Indicates that the medical device can cause discomfort and reddening, irritation, or blistering of the skin	Indicates the Temperature Limit