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Morvabon Desensitizer Gel

Potassium Nitrate 5%

Desensitizer For Hypersensitive Teeth

Morvabon Desensitizer Gel is a stable product which contains fluoride ions. it provides a film like varnish for sensitive teeth, sealing dentinal tubules of exposed dentin and other areas. The product is used either by a dental professional in the office or provided to the patient for home treatment.

Composition:

Sodium Fluoride, Potassium Nitrate, Gel base.

Indications:

- ▶ Quickly eliminates sensitivity from tooth whitening, thermal and chemical changes.
- ▶ Can be used before and after whitening treatment.
- ▶ Relieve the pain of sensitive teeth.

Advantages:

- ▶ Contains potassium nitrate and fluoride ions to reduce the sensitivity.
- ▶ Can be used in home and office whitening treatment.

Instructions for use:

- ▶ place gel in affected areas.
- ▶ While the length of use may vary depending on the condition of the patient, the recommended length of use is between 15 and 60 minutes.
- ▶ After that time, patient should brush and rinse teeth with water to remove all remaining gel.

DO NOT swallow solution during rinsing.

Storage:

- ▶ Store in original container.
- ▶ Replace the cap immediately after use.
- ▶ Keep away from heat and moisture.
- ▶ Do not use after the expiration date.
- ▶ Keep out of reach of children.

Precautions:

- ▶ Only for dental use.
- ▶ In case of consumption drink copious amount of water. Do not induce vomiting. Seek medical advice.
- ▶ Use rubber dam while working with product. Use eye protection, mask and gloves.
- ▶ Do not use Morvabon Desensitizer Gel on patients with reported sensitivity to any of its components.
- ▶ Avoid skin and eye contact to prevent irritation and possible allergic response. If contact with skin or eye occurs immediately remove material with cotton and wash thoroughly with water and seek medical advices.

 Consult instructions for use	 Indicates the batch code	 Catalogue number	 Indicates the date after which the medical device not to be used.	 Indicates a medical device that needs protection from light sources.	 Indicates a medical device that needs to be protected from moisture.
 Do not use if Package is Damaged	 Indicates the medical device manufacturer (Name & Address)	 Indicates a medical device that has not been subjected to a sterilization process.	 Indicates the device intended for use by professional person only.	 Indicates the Temperature Limit	

Produced by Morvabon Co.
 www.morvabon.com
info@morvabon.com
Made in IRAN