ORVABON

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MORVABON.COM

Endocal

(Calcium Hydroxide Paste
with Barium sulfate)

Production
License No.
20708253
IMD: 77389659

ISO 13485: 2016

- ISO13485 : 2016 , US cert, IAS, USA
- Iso 9001: 2015, Tuv certification, Germany
- Iso 14001: 2015, Tuv certification, Germany
- Iso 18001: 2007, Tuv certification, Germany
- Iso 10002: 2014, Tuv certification, Germany
- Iso 10004: 2012, Tuv certification, Germany

Endocal

(Calcium Hydroxide Paste with Barium sulfate)

EndoCal aqueous base radiopaque Calcium Hydroxide paste which is used as a variety purposes material in endodontic treatment. Calcium Hydroxide has been recognized as an antibacterial agent due to its high pH (12.5) as well as the ability to stimulate healing of bone

Advantages:

- ▶ Suitable for temporary filling of infectious canals
- ▶ Stimulates formation of hard tissue in roots and dentine bridges
- ▶ Strong bactericidal effect Up to 72 hours
- ▶ Diffusing Calcium Hydroxide consistently due to its percentage in the product.
- ▶ Ideal viscosity to fill root canals thoroughly
- ▶ Easy to handling and direct application
- ▶ Can be used in conjugation with gutta percha and regular root canal sealants.
- ▶ Shelf life 7-10 days in the mouth.

Method of Use:


- ▶ After root canal preparation, clean and dry canal
- ▶ Attach disposable needle to the syringe and check the flow of paste through the needle
- ▶ Insert the tip into the canal about, stay at least 2mm shy of apex
- ▶ Fill the canal completely by pushing the syringe plunger while slowly withdrawing the tip of syringe
- ▶ Clean excess paste with a sterilized cotton pellet
- ▶ Seal the canal using chosen root canal sealant

Precautions:

- ▶ Apply carefully and replace needle with security cap after use to avoid dryin.
- ▶ For professional dental use only. Avoid contact with skin and eyes.
- ▶ Use in patients who are susceptible to the compounds of this product.
- ▶ Eyes contact: Rinse immediately with plenty of water and consult a physician if burning continues.
- ▶ Skin contact: Rinse thoroughly with plenty of water and consult a physician if it continues to burn or itchy.
- ▶ If swallowed: immediately drink plenty of water and consult a physician.
- ▶ Use rubber-dam while working with the product. It is recommended to use personnel safety equipment (glasses, hats and gloves)

Storage:

- ▶ Protect from direct sunlight and heat.
- ▶ Keep out of reach of children
- ▶ Do not use after the expiration date
- ▶ Store in the original container at room temperature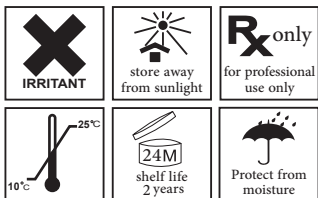

 Produced by Molvabon Co.
www.molvabon.com
info@molvabon.com
Made in IRAN