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MORVABON.COM

Morva Bond Prime

5th Generation Dental
Adhesive (Light Cure)

manufacturer

Pardis Morvabon Pharmed Co.:

Head Office: Unit 154, 7th floor, 7 block,
Borjsaz Bd, Azadi Street, Tehran, Province, Iran

Tel: 0098 21 6658 1438

Fax: 0098 21 6690 8819

- ISO13485 : 2016 , US cert, IAS
- Iso 9001: 2015, Tuv certification
- Iso 14001: 2015, Tuv certification
- Iso 18001: 2007, Tuv certification
- Iso 10002: 2014, Tuv certification
- Iso 10004: 2012, Tuv certification

MEDICAL EQUIPMENT
DEPARTMENT
IMD: 54313039

ISO 13485: 2016

Morva-Bond

5th Generation Dental Adhesive (Light Cure)

It's a light-curing, single component bonding agent for enamel and dentin bonding in combination with the total-etch technique.

Composition:

Bis-GMA, ethanol, HEMA, filler and stabilizers.

Intended use:

- ▶ Adhesive for direct, light curing composite and compomer restorations.
- ▶ Adhesive for indirect all-ceramic and composite restorations (inlays, onlays , veneers)

Advantages:

- ▶ Consistently high bond strength on enamel and dentin
- ▶ Nano-fillers ensure a visible glossy coat which indicates complete coverage
- ▶ Formation of homogeneous layer for improved adhesion to the tooth structure
- ▶ Acetone-free

Precautions:

- ▶ Only for dental use.
- ▶ Wear suitable protective clothing, eyewear and gloves.
- ▶ Do not use on patients with reported sensitivity to any of its components.
- ▶ Carefully read and understand instructions before using. It's necessary to use rubber-dam while working with
- ▶ **Morva-Bond.**
- ▶ Use eye protection, mask and gloves when using **Morva-Bond**
- ▶ Avoid contact with oral soft tissue, eyes or skin
- ▶ If accidental contact with soft tissues, eyes or skin occurs, immediately flush with plenty of water
- ▶ In case of consumption drink copious amount of water. Do not induce vomiting. Seek medical advice
- ▶ Do not inject this product

Warning:

- ▶ Protect from direct sunlight.
- ▶ Keep out of reach of children.
- ▶ Do not use after the expiration date.
- ▶ Store in the original container.
- ▶ This product is designed to be used at room temperature. If stored in cooler, allow product to reach room temperature prior.

- ▶ **Morva-Bond** should be capped immediately after use to minimize evaporation.
- ▶ Do not store product in proximity to eugenol containing products.
- ▶ For cleanup, etchant can be removed with water while the uncured adhesive can be removed with alcohol.

Diresction for use:

- ▶ Isolation (Rubber dam is the preferred method of isolation).
- ▶ Prepare cavity with minimal tooth reduction.
- ▶ Apply etchant to enamel and dentin. Wait 15 seconds. Rinse for 10 seconds.
- ▶ Blot excess water using a cotton pellet or mini-sponge. The surface should appear glistening without pooling of water.
- ▶ Open the cap and squeeze out the exact number of drops you need into the disposable mixing well and then close the cap.
- ▶ Immediately after blotting, apply 2-3 consecutive coats of adhesive to etched enamel and dentin for 15 seconds with gentle agitation using a fully saturated applicator. Gently air thin for five seconds to evaporate solvents. Light cure for 20 seconds.
- ▶ Refer to manufacture's instructions for placement, cure and finishing of restorative materials.

Contraindication:

- ▶ This product is not contraindicated for children, pregnant women, the elderly and other specific diseases and can be used for all people who are not allergic to the active substance

					
Consult instructions for use	Indicates the batch code	Catalogue number	Indicates the date after which the medical device not to be used.	Indicates a medical device that needs protection from light sources.	Indicates a medical device that needs to be protected from moisture.
					
Do not use if Package is Damaged	Indicates the medical device manufacturer (Name & Address)	Indicates a medical device that has not been subjected to a sterilization process.	Indicates the device intended for use by professional person only.	Indicates that the medical device can cause discomfort and reddening, irritation, or blistering of the skin	Indicates the Temperature Limit

Produced by Morvabon Co.
 www.morvabon.com
info@morvabon.com
 Made in IRAN