MORVABON

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Morva-Sealant

Enamel Pit and Fissure sealant

Light curing resin based pit and fissure sealant

Compositions:

- ▶ unsaturated resin, filler, initiator

Advantages:

- ▶ effective protection for the early prevention of caries
- ▶ Low viscosity to flow easily into pits and fissures
- ▶ very economical
- ▶ Fast light curing (12 seconds)

Intended for use:

- ▶ Thoroughly clean teeth to remove plaque and debris from enamel surfaces and fissures.
- ▶ Apply a generous amount of etchant to all enamel surface to be sealed. Etch for a minimum of 15 seconds, but no longer than 60 seconds.
- ▶ Thoroughly rinse teeth with air/water spray to remove etchant.
- ▶ Thoroughly dry the etched surfaces.
- ▶ Apply Sealant into the pits and fissures.
- ▶ cure the sealant by exposing it to light-curing unit for 20 seconds.
- ▶ Wipe the sealant with a sealant with a cotton applicator to remove the thin film on the surface.

Precautions:

- ▶ Carefully read and understand instructions before using.
- ▶ It's necessary to use rubber-dam while working with **Morva-Sealant**.
- ▶ Use eye protection, mask and gloves when using **Morva-Sealant**
- ▶ Avoid contact with oral soft tissue, eyes or skin
- ▶ If accidental contact with soft tissues, eyes or skin occurs, immediately flush with plenty of water
- ▶ In case of consumption drink copious amount of water.
- ▶ Do not induce vomiting. Seek medical advice
- ▶ Do not inject this product
- ▶ For dental use only, Avoid contact with eyes and skin.

Warning:

- ▶ Do not use after expiration date
- ▶ Keep out of reach of children

Contraindication:

- ▶ This product is not contraindicated for children, pregnant women, the elderly and other specific diseases and can be used for all people who are not allergic to the active substance
- ▶ in rare cases the product may cause sensitivity in some people. if any such reaction are experienced, discontinue the use of the product and refer to a physician

Available packaging: 1.2 g syringe

 Consult instructions for use	 Indicates the batch code	 Catalogue number	 Indicates the date after which the medical device not to be used.	 Indicates a medical device that needs protection from light sources.	 Indicates a medical device that needs to be protected from moisture.
 Do not use if Package is Damaged	 Indicates the medical device manufacturer (Name & Address)	 Indicates a medical device that has not been subjected to a sterilization process.	 Indicates the device intended for use by professional person only.	 Indicates that the medical device can cause discomfort and reddening, irritation, or blistering of the skin	 Indicates the Temperature Limit

Produced by Morvabon Co.
www.morvabon.com
info@morvabon.com
Made in IRAN