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Chlorhexidine Gluconate %2 Gel

Green Mint taste and odor Gelgr

Chlorhexidine Gluconate %2 Gel preparation of root canal with mint flavor.

MORVA-Sept %2 is specifically designed endodontic disinfecting purpose only.

This product is more efficient than sodium hypochlorite in destroying such microorganisms as *E. faecalis* who are often responsible for unsuccessful endodontic treatment.

Application:

Chlorhexidine Gluconate %2 Gel facilitates preparation of the root canal.

The lubricants make the introduction of the instrument into the root canal much easier especially recommended.

MORVA-Sept is recommended for endodontic disinfection as a final endodontic rinse, and it should be used after smear layer removal by EDTA.

- ▶ During the endodontic re-treatment, in case of infection caused by enterococcus faecalis or candida albicans.
- ▶ In case of non-healing inflammation of preapical tissues.

Instruction for Use:

- ▶ Insert **MORVA-Sept** gel into the canal using a syringe with applicator or a needle with side portal or directly on the canal instrument for 60 seconds, thereafter during the mechanical preparation of the root canal is recommended.
- ▶ Do not insert **MORVA-Sept** gel beyond the apical foramen.
- ▶ Next rinse the canal with copious amount of Chlorhexidine Gluconate liquid %2 or distilled water.
- ▶ Before using the instrument of another size use gel again, thereafter rinse the root canal.

Caution:

- ▶ Do not mix directly with a NaOCl solution in the canal as a harmful brown precipitate will form.

- ▶ Apply carefully and replace needle with security cap after use.
- ▶ Do not express contents of unit does syringe back into the in dispense syringe.


Advantages:

- ▶ Increase bond strength with dentin bonding agents.
- ▶ Longest and most effective antibacterial activity
- ▶ Ideal viscosity to ensures effective application in root canal
- ▶ A special displacer cap prevents material from being wasted and contaminated.
- ▶ The angled metal tip enables exact application.

MORVA-Sept enables the material be placed and retained in the canal whitout quickly running out.

Warning:

- ▶ For professional dental use only.
- ▶ Avoid contact with skin and eyes.
- ▶ Eye contact: flush eyes with plenty of water. Get innediate nedical attention.
- ▶ Skin contact: flush skin with plenty of water.
- ▶ Ingestion: wash out mouth with water. Give water to drink. Do not induce vomiting.
- ▶ **Keep in cool and dry place.**

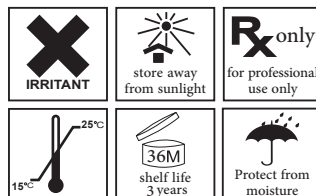Produced by Morvabon Co.
www.morvabon.com
info@morvabon.com
Made in IRAN

MORVA-Sept %2

Chlorhexidine Gluconate %2 Gel

Production
License No.
77340703
IMD: 48828681

ISO 13485: 2016

- ISO13485 : 2016 , US cert, IAS, USA
- Iso 9001: 2015, Tuv certification, Germany
- Iso 14001: 2015, Tuv certification, Germany
- Iso 18001: 2007, Tuv certification, Germany
- Iso 10002: 2014, Tuv certification, Germany
- Iso 10004: 2012, Tuv certification, Germany