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Temporary Cement ZOE TYPE II

Fast Setting Zinc Oxide Cement-Eugenol Base

ZOE is a temporary cement that has analgesic effect upon the tooth and it is used as a dressing or bandage and permanent under noncomposite restoratives. This product to affix dental devices such as crown and bridges. This compound can be applied to protect the tooth pulp.

ZOE is quickly hardened by having a good amount of accelerator material.

The eugenol fluid of high quality and high purity can only be used as an antiseptic, a dental and aesthetic analgesics.

Composition:

- ▶ Powder: Zinc Oxide USP grade, Accelerator 30 gr
- ▶ Liquid: Eugenol USP grade 18 ml

Advantages and Intended use:

- ▶ Temporary cement ideal for teeth bandage or dressing
- ▶ Appropriate protective layer under filled parts.
- ▶ Indirect pulp treatment (with Calcium Hydroxide)
- ▶ Temporary adhesion for dental crowns and bridges
- ▶ Temporary cement for fixed prosthodontics.

Precautions:

- ▶ Carefully read and understand instructions before using.
- ▶ It's necessary to use rubber-dam while working with Zoe.
- ▶ Use eye protection, mask and gloves when using Zoe
- ▶ Avoid contact with oral soft tissue, eyes or skin.
- ▶ If accidental contact with soft tissues, eyes or skin occurs, immediately flush with plenty of water.
- ▶ In case of consumption drink copious amount of water.
- ▶ Do not induce vomiting. Seek medical advice.
- ▶ Do not inject this product.
- ▶ For dental use only.
- ▶ Do not use in similar cases.

warning:

- ▶ Do not use after expiration date.
- ▶ Keep out of reach of children.
- ▶ To maintain the quality of the product and prevent smash, keep it tightly closed and kept in a cool, dry place
- ▶ Keep away from light and moisture in the original package.
- ▶ Store in the original container at room temperature.

Direction for use:


- ▶ place an appropriate quantity of Zinc Oxide powder on mixing pad. Mix in a spatulation cycle about half of the powder with efficient amount of Eugenol and mix fully. The mix must be thoroughly and vigorously spatulated and add remained amount of powder to obtain a smooth creamy mass.

Mixing Time	▶▶▶ 30 Second
Applying Time	▶▶▶ 2 minutes
Setting Time in mouth	▶▶▶ 2-3 minutes

Contraindication:

- ▶ This product is not contraindicated for children, pregnant women, the elderly and other specific diseases and can be used for all people who are not allergic to the active substance.

Produced by Morvabon Co.
www.morvabon.com
info@morvabon.com
Made in IRAN

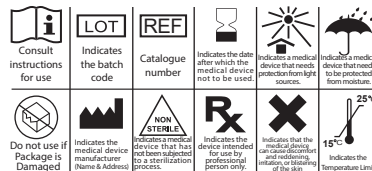TYPE II ZOE

Fast Setting Zinc Oxide
Cement-Eugenol Base

- ISO13485 : 2016 , US cert, IAS
- Iso 9001: 2015, Tuv certification
- Iso 14001: 2015, Tuv certification
- Iso 18001: 2007, Tuv certification
- Iso 10002: 2014, Tuv certification
- Iso 10004: 2012, Tuv certification

Production
License No.
76139023
IMD: 65031875

manufacturer

Pardis Morvabon Pharmed Co.:
Head Office: Unit 154, 7nd floor, 7 block,
Borjzasz Bd, Azadi Street, Tehran, Province, Iran
Tel: 0098 21 6658 1438
Fax: 0098 21 6690 8819
Factory: Unit 135, 1 6, 139, 2nd floor, Seraj Bd.,
20thkm of Damavand Rd., Tehran, Iran
Tel: 0098 21 7625 1702
Fax: 0098 21 7648 1457